

Instructions for cleaning and disinfecting VR TierOne GO headset

Reducing the number of microorganisms by disinfecting the VR device is necessary to maintain the health of users. Clean and disinfect relaxation VR headset between uses.



Prepare: a stock of clean and dry microfiber cloths, disinfectant for quick disinfection of small surfaces with a wide biocidal spectrum (alcohol-based ingredients are allowed) or 75% solution of alcohol, moistened wipes for disinfection, clean water and a paper towel.
You can also disinfect mobile VR goggles quickly and effectively in the UV-C disinfection cabinet.

- 1 Wash your hands thoroughly before cleaning the device.
- 2 Do not use dusty or rough materials for cleaning.
- 3 Remove visible dirt with a cloth dampened with water.
- 4 Do not spray the disinfectant directly onto the device.
- 5 For disinfection, use a cloth moistened with the disinfectant.
- 6 You can also use ready-made disinfectant wipes for disinfection.
- 7 Disinfect the hard surfaces of the device that are most frequently touched.
- 8 Do not use disinfectants on components made of cloth.
- 9 Wipe the lenses with a dry cloth in a circular motion from inside out.
- 10 Clean the VR headset lenses with care to avoid scratching it.
- 11 Use a manual air blower to clean the optics and blow away dust and soiling from the lenses.
- 12 Do not use the alcohol-based disinfectant to clean the lenses.
- 13 Avoid touching the lenses with your fingers while cleaning to avoid greasing the optics.
- 14 Remove and clean the soiled inner cap and replace it with a new one if it is worn.
- 15 Do not tumble dry removable polyurethane leather covers.
- 16 Disinfect the inner PU leather cap and dry it with a paper towel.
- 17 Allow disinfected headsets to dry at room temperature.
- 18 Consider using disposable VR headset masks before reuse.

